

ROM Comparison Grid

	ROM 2.02	ROM 3.02
Test Support	Lipids ALT	Lipids ALT ALT/AST
Change Test Result Order	See attached	Print layout change See Appendix A (attached)
Header Information	Printed only when Framingham was off	The header information: Cholestech, date, time, sample type and name/ID entry, will always be printed on the label when the printer is on, regardless of the other settings.
Sample Type	Not displayed	The sample type, 'Serum' or 'Whole B.' will be printed on the label, in the header under the date.
Test Result Display order		See Appendix B
Print of Date w/month spelled out	9-22-04	22 Sep 2004
VLDL	VLD=20	VLDL no longer calculated
non-HDL (see attached Technical Bulletin #115)	Not calculated	Will be calculated and displayed and printed if applicable. The calculation is TC – HDL = non-HDL
Unit selections	mg/dL&U/L mM/L&U/L	mg/dL mmol/L
Change Error Message	"Temp Too High"	"Temperature Out of Range"
	'Reaction did not occur'	'Reaction did not occur' along with the channel number(s) affected
	'Used Cassette'	Test will fail immediately – 'Used cassette' and the channel(s) that failed will be displayed
Expired cassettes	Test could be performed	No cassette will run if expired, 'WARN 2' message will be displayed. An expired Optics Check Cassette can still run with a 'WARN 2' message after results.
# of print copies option	1 or 2	1, 2, or 3
Print additional copy	A difficult thing to do for users that had Risk assessment off	A user can view previous test results by pressing the 'Data' button, and at the end they will have the option to print an additional copy It is EASIER!
LDL	Result appeared when TRG was < 0.5mmol/L	Will be 'N/A' when TRG < 0.5 mmol/L and > 4.52 mmol/L
Framingham	ATP II version Displayed 'Framingham'	ATP III version Displays 'CHD Risk' There will be a new question that is asked during risk assessment regarding the treatment of blood pressure – 'BP Treated'
Seiko Printer	Supports	Will not support the Seiko printer

If you have any queries, please contact
Point of Care Services Ltd, The Old Dairy, NewHall Farm, Lawshall, IP294 SG
Tel: 01284 830876; Fax: 01284 830286
<http://www.poc-services.co.uk>