

## Removing the ROM Pack and Clip:

1. Turn the Cholestech LDX upside down. Unplug the power supply and printer cable from the Analyzer (Figure A).
2. Locate the white plastic ROM Pack directly beneath the power supply and printer jacks.
3. Lift the ROM Pack Clip up and away from the ROM Pack (Figure B). Note that you may have to break a seal that warns you to unplug the Analyzer.
4. Hold the ROM Pack between your thumb and forefinger and pull straight out (Figure C).

## Installing the ROM Pack and Clip:

*CAUTION: DO NOT plug the power supply into the Analyzer until Step 5.*

1. Slide the ROM Pack into the Analyzer with the label side up and the bottom guides in the tracks (Figure D).
2. Press firmly until the edge of the ROM Pack lines up with the surface of the Analyzer.
3. To secure the ROM Pack to the Analyzer, slide the short end of the ROM Pack Clip into the gap directly beneath the power supply and printer jacks (Figure E) and pull the long end of the clip down over the ROM Pack (Figure F).
5. Plug the power supply into the Analyzer. The following messages will appear on the display:
  - Testing memory
  - Selftest running
  - [BEEP] Config not set
6. The Analyzer will automatically enter the configuration menu. Press the STOP button to exit or scroll through the settings by pressing the RUN button.

Please refer to the User Manual online at [www.cholestech.com](http://www.cholestech.com) for the latest version of the configuration menu and CHD Risk flowchart.

If you experience any difficulty installing the ROM Pack or configuring settings, please call Cholestech Technical Service at (800) 733-0404.



Figure A



Figure B



Figure C



Figure D



Figure E



Figure F

